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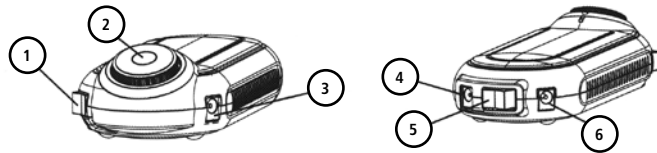
Handstück

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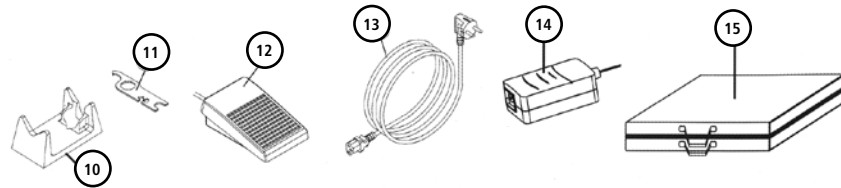
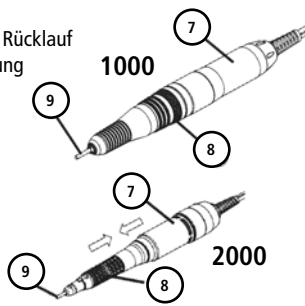
Control Unit

1. Forward / Reverse Selector
2. Speed Control Knob
3. DC Motor Connector
4. Foot Pedal Connector
5. Hand/Foot Selector
6. Battery Charger Connector

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Warning

- Install unit in room at suitable temperature between 0 and 40°C.
- Recycle batteries reg. country rules
- For professional use only
- Use professional bits only
- Remove bit after use and put handpiece carefully onto handpiece stand
- Turn off unit before changing bit

Notice:

Always thoroughly clean Collect Chuck and Bit before reassembling. Turn off unit completely. Never use bent, asymmetrical or damaged bits. Replace after use bit by test bit. Do not leave Handpiece long in break state.

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Operation Procedure

Charging the Battery (Approx. 3 hours to full charge) Connect adaptor DC plug (Fig.14) in DC 15V (Fig.6) on rear of Control Unit. Connect to wall socket 110/220 V. Turn off Speed Control (Fig.2) to charge.

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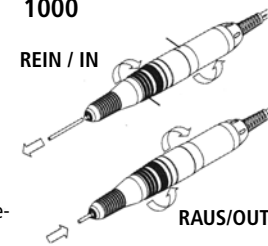
RAUS/OUT

Mounting and Removing of Bit

To remove a bit (9) turn the Chuck Handle clockwise in direction of arrow until it clicks to stop and pull out the bit.

To mount a bit make sure that the chuck is in open position, insert a bit all the way into the chuck, and turn the Chuck Handle clockwise in the opposite direction until it clicks to stop.

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RAUS/OUT

Mounting and Removing of Bit

To remove a bit (9) turn the Chuck Handle clockwise in the opposite direction until it clicks to stop and pull out the bit.

To mount a bit make sure that the chuck is in open position, insert a bit all the way into the chuck, and turn the Chuck Handle clockwise in the direction until it clicks to stop.

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Operating of Unit

Connect DC Motor Cord to „Motor“ (Fig.3) in front of control unit. Turn H/F selector to „H“ (hand) in rear of control unit (Fig.5).

Turn Speed Control Knob (Fig.2) to start motor. Turn off to stop motor. Turn FWD/REV (Fig. 1) to run motor forward or reverse direction. Motor speed can be varied between 0 and 20.000 rpm by Speed Control Knob (Fig.2).

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For using Foot Pedal (Fig.12)

Connect the Foot Pedal plug to the jack marked as FOOT in rear of the control unit. Turn the H/F (Fig.5) to „F“ (foot). Turn speed control knob (Fig.2) . Step on the Foot Pedal to run the motor.

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Safety Protection System

To protect the motor and the control unit from excessive temperature rise caused by overloading such as by failure of ball bearing, greater loading to the bit etc., a temperature sensor activates, when temperature rises above the preset value, to be shut off the power supply. When this functions, turn off the unit. After 10 seconds, restart the unit.

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Typ 2000: 25.000 U/min

Battery Life

Battery (Nickel-Hydrogen) is used in the control unit. Life is limited depending on storage, operation and environment. Take out when not used for a prolonged period to prevent deteriorating. Under normal use of the nit and battery charged once a day, it should last app. 1 year. For replayce it, send to the dealer.

Trouble Shooting

Unit does not run:

Control fully charged, but motor fails to run:

1. Loose connection motor with control unit.
Remedy: Corectly plug the motor cord.
2. Defect in motor cord -Remedy: relace cord
3. Trouble in motor - Remedy: Send unit for repair.

Motor stops suddenly in use:

Overloaded, or Chuck is opened during operation.
Remedy: Check the chuck and close if opened.

Motor fails to run when Foot Pedal is used:

1. H/F Selector is on „H“ Remedy: Set it on „F“.

Motor stopps in 2-3 seconds:

Battery has not been charged Remedy: Charge battery

Motor runs with insufficient power

and stop in a few minutes;Worn battery Remedy:
Replace battery.

Battery recharging fail:

Battery seemed to have been fully charged, motor stops or fails to run

1. Power cord is not connected Remedy: Check at power supply
2. Speed Control Knob is not on „On“
3. blown fuse in AC power adaptor
4. Original power adaptor was not used.

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TROSANI cosmetics GmbH